

# ansa<sup>TM</sup>

a unicare brand

## Raised Toilet Seat with Arms Instruction & Safety Manual

Please read this manual before operation, for  
maintenance instructions and safe usage



The enabling power  
of applied knowledge

Congratulations on your purchase of an ANSA quality product. Before using your product, please read all instructions carefully. The warranty will not apply if the product is used incorrectly. This product has a safe working load of 150kg. Note: Please check that the SWL, on your ANSA product, is applicable to your personal requirements.

## Warranty

Product Warranty - This covers faulty workmanship or defective material for the original consumer purchase for 12 months from the purchase date. In the event of a defect covered by this warranty,

- We will at our option repair or replace the product.
- The item must be returned to your distributor or selling agent with proof of purchase.
- Use and care instructions have been complied with
- No unauthorised repairs or alterations have been made

## Warning

This product has been designed and manufactured with safety and function in mind for general use. For those with special requirements, who may find this product unsuitable, please contact your nearest health professional. This product is not suitable for use as a grab rail and/or stability aid while transferring to, or from the toilet seat.

## Use and Care

- ANSA products are designed for maximum comfort and assistance.
- Heavy and/or uneven user weight distribution may cause stress to the product.
- Ensure you are seated in a central, even position.

## Cleaning and Maintenance

- To clean your ANSA product, use a mild detergent and dry with a clean cloth.
- Regular maintenance inspections are necessary
- Damaged items should be withdrawn from use until rectified.

## Important Information

The following is made in lieu of all warranties, expressed or implied. Sellers and manufacturers only obligation shall be to replace such quality of this product provided to be defective. Neither seller nor manufacturer shall be liable for any injury, loss or damage, direct, indirect or consequential arising out of the inability to use this product. Before using, the user shall determine the suitability of the product for the intended use, and the user will assume all risk and liability whatsoever in connection therewith. No statement or recommendation not contained herein shall have any force or effect unless in an agreement signed by the representatives of the seller and manufacturer.

The patient's nursing care should never be neglected, and when ANSA products are in use, continual assessment should be carried out to ensure the products are still appropriate for the patient's condition. ANSA is renowned for quality products and design. Due to continuous improvement, our products are subject to change without notice.



## Installation

1. Unscrew and remove the existing toilet seat and lid.



2. Unscrew the white nut from the rear crossbar of the Raised Toilet Seat, then insert into in the hole of the toilet
3. Screw the white nut back on to secure the Raised Toilet Seat to the toilet bowl.



4. Push down the raised toilet seat from the front slightly to make it cover the toilet horizontally.





## To adjust the Height

1. Remove the white plug and unfasten the black nut on both sides.
2. Place the crossbar at the required height and screw the black nut back in the left and right hand side of crossbar and fasten securely.
3. Adjust the rear crossbar to the same height as the front.
4. Repeat step 4 of the installation procedure to complete.







A series of 24 horizontal lines for writing, spanning most of the page width.



Lined writing area consisting of 21 horizontal grey lines.



**ansa**<sup>™</sup>  
a unicare brand

PO Box 82 Bentley WA 6982  
T 08 9351 7500 F 08 9350 5299  
[info@unicarehealth.com.au](mailto:info@unicarehealth.com.au)  
[www.unicarehealth.com.au](http://www.unicarehealth.com.au)

Authorised Distributor

