

ansaTM

a unicare brand

Nitro Seat Walker Instruction & Safety Manual

Please read this manual before operation, for
maintenance instructions and safe usage



The enabling power
of applied knowledge

Congratulations on your purchase of an ANSA™ quality product. Before using your product, please read all instructions carefully. The warranty will not apply if the product is used incorrectly.

Note

It is essential to check that the SWL on your ANSA™ product is applicable to your personal requirements.

Warranty

Product Warranty - This covers faulty workmanship or defective material for the original consumer purchase for 12 months from the purchase date. In the event of a defect covered by this warranty.

1. We will at our discretion repair or replace the product.
2. The item must be returned to an authorised ANSA™ distributor or selling agent with proof of purchase.
3. Cost of freight is your responsibility.
4. Use and care instructions have been compiled with.
5. No unauthorised repairs or alterations have been made.

NOTE: Code 6813R Nitro Seat Walker pictured for illustration purpose only. Max user weight: 125kg




WARNING Failure to follow these instructions may result in injury

- Please check the max user weight specified on the SWL Sticker on the Walker Frame is applicable to your personal requirements.
- Do not use your walker until the handles are properly adjusted and securely tightened
- Always lock brakes when sitting on walker
- Do not sit on walker if ground is sloped or uneven
- Do not exceed the maximum weight capacity of the walker
- Do not use this walker as a wheelchair, doing so may cause it to tip-over, resulting in injury




How to fold your walker

Pull the seat handle upwards, then push together the handles until the walker is fully closed.

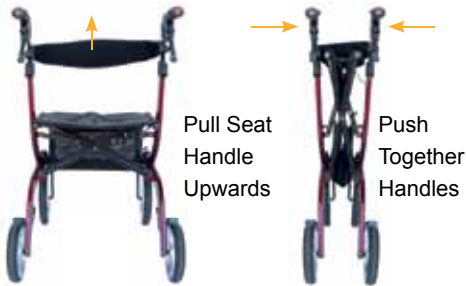

How to Unfold your walker for use

Spread the handles outwards, then push down on seat until the walker is fully open.

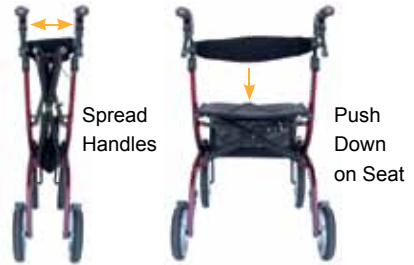

Using the handbrakes



1. To operate the brake pull upwards on the handbrake lever to position (1)
2. When using the walker to move forward, the handbrake lever will remain in position (2)
3. To lock the handbrake, push down on the handbrake lever until it 'clicks' into the lock position (position 2) to position (3)
4. To unlock the handbrake, pull up on the handbrake lever (from position 3) to position (2)

Handle adjustment

1. Remove adjustment knob and bolt from handles
2. Insert handles into walker frame
3. With your arm relaxed at your side, position the handle so the grip is level with the middle of your forearm
4. Insert the bolt through the frame and handle
5. For older style walkers ensure, before tightening the adjustment knob, that the hexagonal bolt head is into the hexagonal hole in the frame. With new style walkers the receiver thread is part of the walker frame
6. Tighten knob so handles are firm

Brake tension adjustment

1. To adjust cable at the handbrake handle - 1. Loosen lock nut. 2. Turn cable adjuster clockwise to loosen the brake. 3. Turn cable adjuster anti-clockwise to tighten the brake
2. If further adjustment is required please contact your Unicare Health product distributor

Cleaning & Maintenance

Keep hair and lint from around wheel axles and bearings and oil moving parts occasionally. Ensure brakes are at optimum adjustment and handle adjustment knobs are tight. If walker frame is bent or shows sign of fatigue DO NOT USE WALKER.



ansa[™]
a unicare brand

PO Box 82 Bentley WA 6982
T 08 9351 7500 F 08 9350 5299
info@unicarehealth.com.au
www.unicarehealth.com.au

Authorised Distributor

